

Instruction for Use A (WS-COV30PN-2)



- Step-A-1**
The Needle/Stylet(15G) is inserted into the epidural space of the spine through Hiatus above the coccyx.
- Step-A-2**
Pull out the Stylet and place the Guide Wire through the Needle.
- Step-A-3**
Pull out the Needle leaving the Guide Wire in place.
- Step-A-4**
Insert Dilator over the Guide Wire into the hiatus.
- Step-A-5**
Remove the Dilator and put Sheath/Stylet onto Guide Wire. Pull out Guide Wire and Stylet. Leave the Sheath in the hiatus

Instruction for Use B (WS-COV30PN-5)



- Step-B-1**
The Introducer/Needle (8G) is inserted into the epidural space of the spine through Hiatus above the coccyx.
- Step-B-2**
Pull out the Needle. Leave the Introducer in the hiatus.

- Insert a flexible endoscope inside of Video Guide Epidural Catheter until it can get to the end of the Catheter's Hub. Fill up Catheter using a prefilled saline syringe.
- Insert the Catheter into the targeted treatment point through the Sheath or the Introducer
- Check where the Catheter is placed with C-Arm. Inject the mixed medicine into the Catheter through the injection hole of the body. Once the procedure is completed, remove the Catheter and accessories.


SPECIO

Video Guide Catheter

- Diagnosis and surgical procedure in the epidural space
- Minimally invasive
- Rapid recovery time

Video Guide Catheter

The SPECIO is an infusion device for Percutaneous Epidural Neuroplasty and can be used with a fiberoptic endoscopy . This is a medical device which is made for delivering injected medicine into the suspect spot of epidural space.

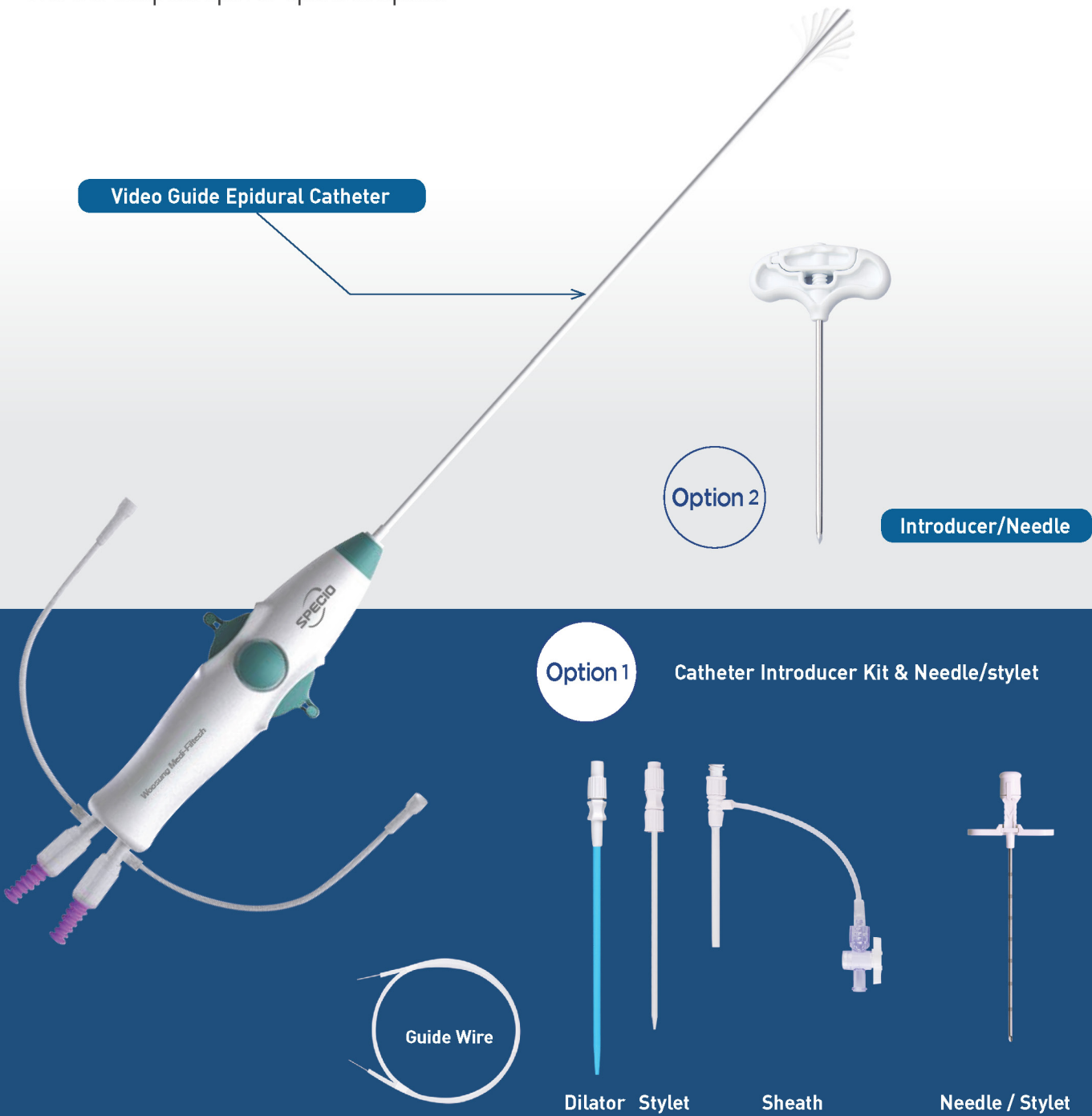

Indications

- Chronic Back Pain
- Post- Laminectomy Syndrome
- Spinal Stenosis
- Herniated Nucleus Pulposus

Contraindication

- Pregnancy
- Infection
- Metal illness

Feature

- Lightweight, ergonomic design
- Catheter outer diameter is 3mm
- Soft, atraumatic and radiopaque tip
- Radiopaque shaft
- Dual working channels
- Dual infusion ports with rubber buffers
- Direct visualization via endoscope
- 2-way steering
- Locking/unlocking button

Ordering Information

No.	Article Name	Component Article Name	Device Name	Q'ty(ea)
1	WS-COV30PN-2	WSV30P-2	Video Guide Epidural Catheter	1
		WSVP-30V	Catheter Introducer Kit	1
			Dilator	
			Guide Wire	
2	WS-COV30PN-5	WSVP-30A	Needle / Stylet	1
		WSV30P-2	Video Guide Epidural Catheter	1
		WSVP-8V	Introducer/Needle	1